



## WhaleTeq SECG 4.0 Software

Version: 5.0.9.16

=====

1. Enhanced function of Auto-HR /Auto-Pacing
2. SECG assistant: IEC60601-2-27 test item 201.12.1.101.1 modify. Add steps, let the user input the full scale number before starting the test
3. SECG assistant: IEC60601-2-27 test item 201.12.1.101.2 modify. Add test Waveform, Triangle or Sine
4. SECG assistant: IEC60601-2-27, test item 201.12.1.101.16 modify. Add steps, let the user input the full scale number before starting the test
5. SECG assistant: IEC60601-2-25, 201.12.4.107.1.1.1 modify. Add Method B) C) D)
6. SECG assistant: IEC60601-2-25 test item 201.12.4.108.3.2 modify. Method B: triangle pulses 1 mV, 50 ms, 120 BPM
7. SECG assistant: IEC60601-2-25 test item 201.12.4.109 modify. Test 1: Pacing add "Off (Vref)" (Pulse Amplitude is zero). Test 2: add "Pulse, 2 mV, width 100 ms"
8. Calibration mode: Variable DC Offset Limit from 5% modify to 1%